

The Solution for Worry-Free Control of Patient Belongings

OP/ER Valet Locking Pouch



Easily attaches to any stretcher



Locks for worry-free belonging security



Cleaned with same protocol used on mattresses



“The best non-clinical decision our Emergency Department ever made.”

– Martha Herman, R.N.



OP/ER Valet Nylon Liner



Sanitary single-use liner



Tamper-evident self-sealing liner



Patient belongings quickly and easily stored

Custom built to be the perfect fit for your specific needs

OP/ER Valet™ **For Worry-Free Control of Patient Belongings**

Worry-Free Security

- Virtually eliminates the risk of a patient's belongings being misplaced or stolen while in the ER or out-patient setting.
- Key is kept only by approved hospital staff.
- Easily labeled with patient's ID.

Easy to Use and Sanitary

- In just a few easy steps the bag is attached to any stretcher.
- Patient's belongings are sealed in the bag, locked away and safely secured.
- Tamper-evident, single-use liners promote infection control.

Space-Saving & Durable

- Eliminates the need for costly lockers that also consume valuable space.
- Heavy-duty construction ensures a long life of worry-free use



Custom built to be the perfect fit for your specific needs

Using the OP/ER Valet™

1. When a patient who has been admitted into your department needs to disrobe for a procedure, the attending nurse will issue the patient one OP/ER Valet™ single use, sanitary, tamper evident liner.
2. After the patient disrobes, the nurse will place their belongings into the liner and inventory, expose the adhesive strip, and seal the liner. Now their belongings are sealed, much like an evidence bag.
3. Using the easy carry handle, the nurse will insert the liner into the pouch, zip it closed, and lock the pouch.
4. Once the patient is ready for discharge, the patient will hand the key to the attending nurse who will unlock the pouch and remove the sealed liner. The nurse will visually inspect the liner to insure it is still sealed and has not been tampered with. The patient is now ready to change and leave



Custom built to be the perfect fit for your specific needs

Attaching the OP/ER Valet™ pouch to a stretcher:

1. Determine the position on the stretcher for desired access.
2. Lift the mattress, remove adhesive backing from the Velcro on the pouch and attach to stretcher.
3. Press firmly to make sure adhesive backing on Velcro is secure.
4. Additional stability can be achieved by bolting through the grommets in the flap or utilizing a lanyard to secure the pouch.
5. You have now created a secure and tamper evident system for patient belongings in your department.



Custom built to be the perfect fit for your specific needs

Rack, wall mount: Part# 54287-001

Wall mounting hardware included

Liners (100/box, 600/case) Part# 53850-001

Wall rack holds one box o 100 OP/ER Valet™ liners

The OP/ER™ Valet wall mount rack makes it quick and easy to dispense liners for patient belongings. The rack attaches with two screws to any flat surface capable of holding 12 pounds.



Custom built to be the perfect fit for your specific needs



**OP/ER Valet
XL Pouch with Lock**
Part # 56466-001

Custom built to be the perfect fit for your specific needs